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OXYGENAL 6 – 0.489.3451 (ref. Roam 103318)

User instructions

KaVo and OXYGENAL 6 are either registered trademarks or trademarks of KaVo Dental GmbH. All other trademarks are property of their respective owners. OXYGENAL 6 can be used in all dental treatment units from KaVo Dental GmbH. OXYGENAL 6 is not compatible with dental treatment units not produced by KaVo Dental GmbH.

KaVo Technical Service

If you have any questions or complaints, please contact the KaVo Technical Service: +49 (0) 7351/56-2500 or service.treatmentunits@kavo.com

Intended user and Use Environment

Professional users such as dentists and dental practice personnel in a healthcare environment.

General marks and symbols

	Medical Device, labelling of medical devices
	Follow the (electronic) instructions for use
	Store in upright position
	Storage temperature: +5°C to +30°C
	Store out of direct sunlight
	Keep dry
	Adhere to the expiry date
	Reference number
	Unique Device Identifier
	CE-mark for medical device (1639 = Notified Body Number)
	Manufacturer
	Production batch
	Note: Please note accompanying documents
	Distributor
	Legal Representative in Switzerland

Transport and Storage

OXYGENAL 6 solution should be stored in the original bottle at a temperature between +5° C and +30° C. Keep away from sunlight. Keep dry and store in upright position. If stored under the appropriate conditions, the product has a shelf-life of two years. Keep bottle closed until use. Due to the design of the cap intended to relieve pressure in the bottle, the bottle may seem to leak if it is kept in wrong storage conditions. The bottle can be used up to 4 months after opening.

Currently applicable packaging law

Dispose of and recycle the packaging appropriately in accordance with current packaging law, employing waste management/recycling companies. Do not reuse empty bottle for other purposes. Rinse and dispose as per local regulations.

Warnings

WARNING

Before using the product, read thoroughly the safety data sheet. This can be obtained through the local distributor. Do not use OXYGENAL 6 if the packaging has been damaged or has been exposed to incompatible storage conditions.

Causes serious eye irritation. Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice. Contact with OXYGENAL 6 may discolor skin. Wipe up immediately if spilled.

NOTE

All serious events occurring in relation to the medical device must be reported to the manufacturer and the competent authority of the Member State, in which the user and/or patient resides.

The instructions for use are a component of the product and must be read carefully prior to use and must be accessible at all times. The medical device may only be used in accordance with the intended use, any other type of use is not permitted.

UFI: XQVH-JOWJ-WOOW-WERM

Product Description

Intended Purpose

OXYGENAL 6 is intended for germ reduction and prevention of biofilm in dental unit water lines of KaVo treatment units.

OXYGENAL 6 is only intended to be used in dental chairs manufactured by Kavo Dental GmbH.

Follow the KaVo care instructions for the specific dental treatment unit: handling also depends on the equipment option. The KaVo care instructions and electronic IFU for the dental treatments units and its accessories (among which OXYGENAL 6) can be downloaded on www.kavo.com/download. The user must observe the directions for use, as non-compliance may affect the safety and performance of the medical device.

Indications

OXYGENAL 6 is not intended to be used on a specific patient population, it is intended as a risk control measure for dental units during patient care.

Contra-indications

OXYGENAL 6 should not be used outside a healthcare setting.

Composition

Contains hydrogen peroxide (CAS 7722-84-1): 6%.

Made in Belgium

Roam Technology NV - I.Z. Poort Genk 6835 - Geleenlaan 24 - 3600 Genk - Belgium
T: +32(0)89 44 00 42 - info@roamtechnology.com

KaVo Dental GmbH - Bismarckring 39
88400 Biberach an der Riß - Germany - T: +497351560

OXYGENAL 6 – 0.489.3451 (ref. Roam 103318)

User instructions

KaVo and OXYGENAL 6 are either registered trademarks or trademarks of KaVo Dental GmbH. All other trademarks are property of their respective owners. OXYGENAL 6 can be used in all dental treatment units from KaVo Dental GmbH. OXYGENAL 6 is not compatible with dental treatment units not produced by KaVo Dental GmbH.

KaVo Technical Service

If you have any questions or complaints, please contact the KaVo Technical Service: +49 (0) 7351/56-2500 or service.treatmentunits@kavo.com

Intended Users and Use Environment

Professional users such as dentists and dental practice personnel in a healthcare environment.

General marks and symbols

	Medical Device, labeling of medical devices
	Follow the (electronic) instructions for use
	Store in upright position
	Storage temperature: +5°C to +30°C
	Store out of direct sunlight
	Keep dry
	Adhere to the expiration date
	Reference number
	Unique Device Identifier
	CE-mark for medical device (1639 = Notified Body Number)
	Manufacturer
	Production batch
	Note: Please note accompanying documents
	Distributor
	Legal Representative in Switzerland

Transport and Storage

OXYGENAL 6 solution should be stored in the original bottle at a temperature between +5°C and +30°C. Keep away from sunlight. Keep dry and store in upright position. If stored under the appropriate conditions, the product has a shelf-life of two years. Keep bottle closed until use. Due to the design of the cap, intended to relieve pressure in the bottle, the bottle may seem to leak if it is kept in wrong storage conditions. The bottle can be used up to 4 months after opening.

Currently applicable packaging law

Dispose of and recycle the packaging appropriately in accordance with current packaging law, employing waste management/recycling companies. Do not reuse empty bottle for other purposes. Rinse and dispose of as per local regulations.

Warnings

WARNING

Before using the product, carefully read the safety data sheet. This can be obtained through the local distributor. Do not use OXYGENAL 6 if the packaging has been damaged or has been exposed to incompatible storage conditions.

Causes serious eye irritation. Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice. Contact with OXYGENAL 6 may discolor skin. Wipe up immediately if spilt.

NOTE

All serious events occurring in relation to the medical device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient resides.

The instructions for use are a component of the product and must be read carefully prior to use and must be accessible at all times. The medical device may only be used in accordance with the intended use; any other type of use is not permitted.

UFI: XQVH-JOWJ-WOOW-WERM

Product Description

Intended Purpose

OXYGENAL 6 is intended for germ reduction and prevention of biofilm in dental unit water lines of KaVo treatment units.

OXYGENAL 6 is only intended to be used in dental chairs manufactured by Kavo Dental GmbH.

Follow the KaVo care instructions for the specific dental treatment unit; handling also depends on the equipment option. The KaVo care instructions and electronic IFU for the dental treatments units and its accessories (among which OXYGENAL 6) can be downloaded on www.kavo.com/download. The user must observe the directions for use, as non-compliance may affect the safety and performance of the medical device.

Indications

OXYGENAL 6 is not intended to be used on a specific patient population, it is intended as a risk control measure for dental units during patient care.

Contra-indications

OXYGENAL 6 should not be used outside a healthcare setting.

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